


ClearliQ

Earwax Drops

Instructions for use

 Carefully consult the instructions for use, even if you have used the product before. Do not dispose of the instructions for use as it contains important information which may have been amended according to new research and findings since you last used the product. Store the instructions for future reference.

ClearliQ Earwax Drops

Consist of:

- ClearliQ Earwax Drops 10ml e
- Instructions for use
- Consumer box

Earwax, also known as cerumen, is a natural secretion produced by your ear. It is made up of various components like fatty acids, alcohols, squalene, and cholesterol. In case of excessive earwax, it can block the ear canal and may lead to discomfort, hearing loss, or even inflammation.

Intended use

ClearliQ Earwax Drops is designed for softening excessive earwax, suitable for adults and children from 2 years.

Mode of action

The mode of action of ClearliQ Earwax Drops involves a rich, low viscosity, oil-based formula that softens, and dissolves hardened earwax. This process makes earwax easier to remove and prevents the formation of hard earwax plugs. The drops also serve as an effective pre-treatment for ear syringing, ensuring that earwax can be

safely and effectively removed, whether at home or by a healthcare professional.

Directions for use

Ensure hands are clean before handling the product. Warm the formula by holding the bottle in your hands for a few minutes to bring the product to body temperature. Shake the bottle gently before use. Tilt your head to one side, apply the tip near the entrance of the ear canal without inserting it. Apply 3-5 drops, on the earwax in the affected ear. Remain in the same position for a few minutes allowing the formula to penetrate the earwax. Gently massage the base of the ear to help dissolve the earwax. The formula can reach the earwax and inner ear more effectively by gently pulling the earlobe to the side. Sit up after a few minutes and wipe away any excess liquid and softened earwax with a clean tissue. Do not insert any objects into the ear canal to remove earwax. Ensure to close the cap of ClearliQ Earwax Drops after each application to prevent the formula from drying out.

Dosage

Apply 3-5 drops on the earwax, per ear twice a day, for a maximum of 6 consecutive days. Unless otherwise advised by a doctor or specialist.

Precautions for use

Do not use in case of:

- Damaged or perforated eardrums, grommets, (ear tubes), open wounds (for

example, after ear surgery) or when the ear canal is blocked by a foreign object.

- When you are hypersensitive or allergic to any of the ingredients.
- Consult a doctor if you experience pain, persistent irritation, or an allergic reaction.
- Consult your doctor or pharmacist before using this product if you are pregnant or breastfeeding or when you suffer from an ear infection.
- ClearliQ Earwax Drops is a product intended for single-patient use.
- For external use only.

Storage

- Store in a cool, dry place away from direct sunlight.
- Keep out of reach of children.
- ClearliQ Earwax Drops can be used 12 months after opening.
- When the treatment is complete dispose the bottle, leaflet and carton as with any common urban waste. Consider the environment and recycle.
- Lot number and expiration date: see bottle.

Ingredients

Seed oils (Vitis Vinifera Seed Oil, Brassica Campestris Seed Oil) 82,5%, Squalane 12%, DBE (Dimethyl Glutarate, Dimethyl Adipate, Dimethyl Succinate) 5%, Tocopheryl Acetate.

Possible side effects

If used according to the instructions for use, this product is well tolerated. Regular or persistent side effects are

not known so far. If a serious incident has occurred in relation to the product, report to the manufacturer mentioned on the leaflet and the competent authority in your country.

Key symbols



See instructions for use



English text Symbol



Keep away from sunlight



Product is intended single patient use



Medical device



CE Marking



Unique device identifier



Lot number and expiration date



Product must be stored between 10° and 30° Celsius



Product can be used 12 months after opening

ClearliQ



Manufacturer
NVV Pharma B.V.
Keizersgracht 62
1015CS Amsterdam
The Netherlands



Distributor
(Distributor name)
(Address)
(Postal code + City)
(Country)