


# ClearliQ


# Earwax Spray

Instructions for use

 **EN** Carefully consult the instructions for use, even if you have used the product before. Do not dispose of the instruction for use as it contains important information which may have been amended according to new research and findings since you last used the product. Store the instructions for future reference.

### ClearliQ Earwax Spray

Consist of:

- ClearliQ Earwax Spray 12ml 
- Instructions for use
- Consumer box

Earwax, also known as cerumen, is a natural secretion produced by your ear. It is made up of various components like fatty acids, alcohols, squalene, and cholesterol. In case of excessive earwax, it can block the ear canal and may lead to discomfort, hearing loss, or even inflammation.

### Intended use

ClearliQ Earwax Spray is designed for softening and removal of excessive earwax, suitable for adults and children over 2 years and older.

### Mode of action

The mode of action of ClearliQ Earwax Spray involves a rich oil-based, low viscosity formula that transforms excessive hardened earwax into soft earwax, the formula works 4 in 1.

1. Softens and liquefies earwax
2. Dissolves earwax (plugs)
3. Clears the ear canal
4. Relieves discomfort

### Directions for use

Ensure hands are clean before handling the product. Twist the nozzle horizontally. Before using the bottle for the first time, prime the pump by pressing it 3-4 times to remove any air. Warm the formula by holding the bottle in your hands for a few minutes to bring the product to body temperature. Shake the bottle gently before use. Tilt your head to one side, apply the nozzle tip near the entrance of the ear canal. Apply 2-3 sprays, on the earwax, in the affected ear. Remain in the same position for a few minutes to allow the liquid to penetrate the earwax. Gently massage the base of the ear to help to dissolve the earwax. The formula can reach the earwax and inner ear more effectively by gently pulling the earlobe to the side. Sit up after a few minutes and wipe away any excess liquid and softened earwax with a clean tissue. Do not insert any objects into the ear canal to remove earwax. After use, clean the spray head with warm water and dry with a lint-free cloth. If the spray begins to weaken - often due to not being stored upright - restore its function by pumping the spray several times while holding the bottle upright.

### Dosage

Apply 2-3 sprays on the earwax, per ear, twice a day, for a maximum of 6 consecutive days. Unless otherwise advised by a doctor or specialist.

### Precautions for use

Do not use this product in case of:

- Damaged or perforated eardrums, grommets, (ear tubes), open wounds (for example, after ear surgery) or when the ear canal is blocked by a foreign object.
- When you are hypersensitive or allergic to any of the ingredients.
- Consult a doctor if you experience pain, persistent irritation, or an allergic reaction.
- Consult your doctor or pharmacist before using this product if you are pregnant or breastfeeding or when you suffer from an ear infection.
- ClearliQ Earwax Spray is intended for single-patient use.
- For external use only.

### Storage

- Store in a cool, dry place away from direct sunlight.
- Keep out of reach of children.
- ClearliQ Earwax Spray can be used 12 months after opening.
- When the treatment is complete dispose the bottle, leaflet and carton as with any common urban waste. Consider the environment and recycle.
- Lot number and expiration date: see bottle.

### Ingredients

Seed oils (Vitis Vinifera Seed Oil, Brassica Campestris Seed Oil) 82,5%, Squalene 12%, DBE (Dimethyl Glutarate, Dimethyl Adipate, Dimethyl Succinate) 5%, Tocopheryl Acetate.

### Possible side effects

If used according to the instructions for use, this product is well tolerated. Regular or

persistent side effects are not known so far. If a serious incident has occurred in relation to the product, report to the manufacturer mentioned on the leaflet and the competent authority in your country.

### Key symbols



See instructions for use



English text Symbol



Keep away from sunlight



Product is intended single patient use



Medical device



CE Marking



Unique device identifier



Lot number and expiration date



Product must be stored between 10° and 30° Celsius



Product can be used 12 months after opening

# ClearliQ



**Manufacturer**  
NVV Pharma B.V.  
Keizersgracht 62  
1015CS Amsterdam  
The Netherlands



**Distributor**  
(Distributor name)  
(Address)  
(Postal code + City)  
(Country)

Leaflet: last updated:  
03 December 2025  
E04014XX101